

1



Canine sedated\* and in Lateral recumbency.

\*Sedated: in accordance with clinician consideration (animal behaviour)

2

Medial Proximal Tibia of recumbent hind limb locally sedated, clipped, and/or aseptically prepared.



3

Index finger (2<sup>nd</sup> Digit) palpating patellar ligament. Sliding down to rest on Proximal Anterior aspect of Tibial Tuberosity.



4

Penetration site: Thumb (1<sup>st</sup> Digit), palpating the flat surface of the Medial Proximal Tibia.



### Alternative Method:

1

Draw a line Perpendicular to Tibial Crest, mid-level.




2

Draw a line at 90° in the middle of the previous line (Mid Tibial).



3

Penetration Site.

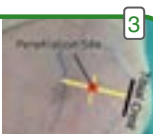

**How to  
use the  
Vet B.I.G.<sup>™</sup> ?**  
Brief Guidelines

# How to use...

Adjust to an appropriate penetration depth.



Hold firmly the bottom part, at 90° to the leg.



Squeeze and pull out the safety latch.



While holding firmly at the bottom part, press down with the palm of your hand.



Pull up VetBIG slowly.



Remove the Trocar needle.



Secure with the safety latch.



WaisMed Ltd.  
Leading Intraosseous Devices

[www.waismed.com](http://www.waismed.com)

[mail@waismed.com](mailto:mail@waismed.com)