



PENETRATION SITE:

From Tibial Tuberosity (A) go approx. 2 cm toward inner leg (B) and approx. 1 cm toward knee (C)

2

Hold firmly the bottom part, at 90° to the leg



3

Squeeze and pull out the safety latch



4

While holding firmly at the bottom part, press down with the palm of your hand



5

Pull up the BIG slowly



6

Remove the trocar needle



7

Secure with the safety latch



8

Flush with 10-20 cc of normal saline



Use aseptic technique throughout!

**How to
use the
Adult BIG?**
Brief Guidelines

What if...

1

Trocar Needle is stuck after triggering?

Reason: Inaccurate penetration site.

WHAT TO DO?

Match safety latch square hole to the hub.

Pull needle out gently.

Find the right penetration site on the other leg.

Use a new BIG.

2

Needle is not stable?

Reason: Unfirm hold and/or wrong angle positioning.

WHAT TO DO?

Pull needle out.

Find the right penetration site on the other leg.

Use a new BIG, while holding firmly at 90° to the leg.

3

Needle is in place, but there is no flow?

Reason: cannula is blocked.

WHAT TO DO?

Flush with a syringe up to 20cc of normal saline and proceed with infusion.

rev. 02/08



WaisMed Ltd.
Leading Intraosseous Devices

www.waismed.com

mail@waismed.com