



PENETRATION SITE:

From Tibial Tuberosity (A) go 1-2 cm toward inner leg (B) and 1-2 cm down toward foot (C)


Adjust depth according to patient's age



Hold firmly the bottom part, at 90° to the leg



Squeeze and pull out the safety latch

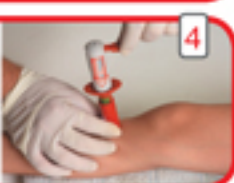

While holding firmly at the bottom part, press down with the palm of your hand



Pull up the BIG slowly



Remove the trocar needle



Secure with the safety latch



Flush with 5-10 cc of normal saline



Use aseptic technique throughout!

How to use the Pediatric BIG?
Brief Guidelines

What if...

1
Trocar Needle is stuck after triggering?

Reason: Inaccurate penetration depth and/or penetration site.

WHAT TO DO?

Match safety latch square hole to the hub.

Pull needle out gently.

Find the right penetration site on the other leg.

Use a new BIG after adjusting the depth according to patient's age.

2
Needle is not stable?

Reason: Unfirm hold and/or wrong angle positioning.

WHAT TO DO?

Pull needle out.

Find the right penetration site on the other leg.

Use a new BIG, while holding firmly at 90° to the leg.

3
Needle is in place, but there is no flow?

Reason: cannula is blocked.

WHAT TO DO?

Flush with a syringe up to 10cc of normal saline and proceed with infusion.

rev. 02/08



WaisMed Ltd.
Leading Intraosseous Devices

www.waismed.com

mail@waismed.com